

Gypsum Solvent



Dental

Easily cleans

- mixing bowls
- acrylic dentures
- impression trays
- cast objects

from traces of dental plasters and plaster based investment materials, even in the most difficult places.

Use:

Place the object to be cleaned in the undiluted solution.

The cleaning process is speeded up by using an ultrasonic bath and heating the bath up to 40 – 50 °C.

Cleaning time:

approx. 20 – 80 min.

Wash the object in water after removing it from the bath. The cleaning power of the solution can be seen by the formation of bubbles.

1000 ml of the solution dissolve approximately 100 g of plaster completely.

Shelf life

12 month